

Setting  
POWER  
Custom Made  
REHAB  
Sports & Rec  
Innovation

# Solara® 2G Limited



The Invacare® Solara® 2G Limited wheelchair offers innovative weight-shift tilt-in-space technology for ease of tilt, along with Smart Adjustability™, portability and proven reliability. The Solara 2G Limited is a giant step forward in design, aesthetics, functionality and most importantly, benefit to the consumer.



## The Finer Details

- Seat-to-floor heights starting as low as 14" accommodate standing pivot transfers and/or foot propellers
- Exceptional ground clearance achieved even at lowest seat-to-floor heights
- Easy to read tilt angle indicator even works on an unlevel surface
- Flared front end provides more room for the user's feet
- Telescoping rigging receivers (medium / large frames) are double-bolted for increased durability
- Adjustable depth, adjustable angle back canes are standard
- One axle plate accommodates all wheel options
- Anti-tippers included with base chair are designed to remain out of way while caregiver pushes the chair
- Stealth® headrests and Invacare® Seating Options available from Stealth Products

## Solara® 2G Limited Specifications

Frame style	Tilt
Seat width	16" - 20"
Seat depth	15" - 19"
Back height	20" and 24"
Overall chair width open	Seat width plus 8.5"
Tilt range	-5° to 50°
Camber angles	0°
Tubing outside diameter	1" Reg, Seat rail 1"
Weight capacity	250 lb.
Weight of chair	28.5 lb. ***/41.5 lb. ****
Limited warranty	Lifetime on frames and crossbraces

\*\*\* Base chair weight without accessories

\*\*\*\* 18" x 18", FXDESK, BH20, FLDBACK, AT902, 12COM, A1577, PNEU, 70NTAPER, 1651, TLC, Anti-tippers, Wheel Locks

## Knee-to-Heel Range

(Range adjusts in 1" increments. Measurement taken from top of seat rail to back of footrest)

Front rigging type	min	max
60°	13.5"	17.5"
70°	13.5"	16.5"
70° tapered	12.75"	16.25"
Elevating	14"	20"

## Arm Heights

(All measurements from top of seat rail to top of armrest)


Adjustable-height-dual-point 8.5" - 12.5"  
(conventional only, range adjusts in 1" increments)

## Front and Rear Seat-to-Floor Heights

(Measured from floor to top of seat rail)

Rear Wheel	5" Caster 5SOF <sup>T</sup> 5ALUM	6" Caster AT902	6" Caster 1259	8" Caster 1228 AT903
12"	16.00"	16.00"	16.00"	17.00"
	16.50"	16.50"	16.50"	17.50"
	17.00"	17.50"	18.00"	
18"	14.50"	15.50"	16.00"	18.50"
	15.00"	16.00"		19.00"
	15.50"	16.00"		
20"	15.00"	15.50"	16.00"	17.00"
	15.50"	16.00"	16.50"	
	16.00"	16.50"	17.00"	
22"	15.00"	16.00"	16.00"	17.00"
	15.50"	16.50"	16.50"	17.50"
	16.00"	17.00"	17.50"	18.00"
24"	15.00"	16.00"	17.00"	17.00"
	15.50"	16.50"	18.00"	17.50"
	16.00"	N/A	18.00"	18.50"
				19.00"

All dimensions are approximate

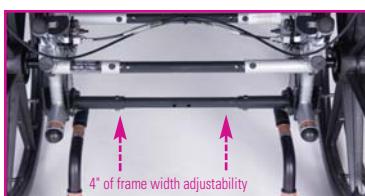

A user's weight may change between fitting and delivery. Solara 2G Limited offers 4" of frame width adjustability, allowing fine tune adjustment in the field.



Recline mechanisms are now mounted on the outside of the back canes to provide room for off-the-shelf back hardware. 170° of recline is available with this option.



The Solara 2G Limited has proven gear teeth mechanisms and its gear rack concept revolutionized the durability of tilt-in-space wheelchairs.



New dual-point arms have height adjustability and flip back easily for transfers.



Tilt lock-out stops may be used to limit the tilt range on the low-end, high-end or both.



Improved tilt angle indicator and minimized rear hardware for easy seat depth adjustment.

Invacare Canada LP  
www.invacare.ca

Canada  
570 Matheson Blvd. E.  
Unit 8  
Mississauga, Ontario  
L4Z 4G4 Canada  
(800) 668-5324

All rights reserved.  
Trademarks are  
identified by the  
symbols <sup>TM</sup>, <sup>SM</sup> and  
®. All trademarks are  
owned or licensed to  
Invacare Corporation  
unless otherwise noted.

Specifications  
are subject to change  
without notification.

© 2008  
Invacare Canada  
Form No. 08-666C